

SWISS WHITE

TOOTH WHITENING GEL

With Fluoride

Composition

Syringe 3.3g tooth whitening gel contain 10% or 16% or 22% of carbamide peroxide. SWISSWHITE is a tooth whitening gel with fluoride.

Properties & Indications

SWISSWHITE is a bleaching gel for whitening vital and non vital teeth using a custom made tray for home bleaching under dentist supervision

Directions for use

-Do not use if primary package is broken. -Do not use on a person under 18 years age. -Dentist regulates the treatment and determines how long it should last. -SWISSWHITE should be sold only to dental practitioners. -First use to be only done by dental practitioner or under their direct supervision, after that consumer can complete the cycle of use.

1. Insert a small amount of gel in the hollow tooth area of the bleaching tray. Do not fill with too much bleaching gel, as this does not improve or speed up the whitening process. One syringe is adequate for 8 to 10 applications. Once open

syringes can be stored closed with syringe cup. 2. Insert the tray with bleaching gel into the mouth and place over the teeth. Do not press the tray. 3. Remove excess material from the gums or palate. 4. Wear the bleaching trays for at least 2 hour day or overnight as instructed by your dentist. 5. Do not eat, drinks smoke while wearing the bleaching tray. 6. Remove the tray after procedure. Clean your teeth and rinse with water thoroughly to remove any residuals of gel. Clean

tray thoroughly with warm water and a toothbrush. Dry it properly and store in the storage box. 7. Repeat the procedure as instructed by your dentist.

Note, avoid highly acidic food and drink (cola, citrus drinks and fruit), as these can cause hypersensitivity of the tooth and oral mucosa. Strongly colored drinks (coffee, tea, red wine) as well as smoking should be avoided during the whole procedures period, as these can retard the whitening process. Definite whitening can be normally seen after approximately 5 days. In the case of stubborn staining, your dentist could extend the period whitening procedures considerably.

Depending on the tooth structure and habits, whitening lasts for several years. You can repeat the procedure later without any damage on the tooth.

Tooth filling, crowns, and veneers (porcelain, composite and metal) aren't bleached. Two weeks time should be between bleaching and renewing restorations.

In certain cases, if mucosal irritation or teeth hypersensitivity may appear to hot or cold, during the bleaching, these disappear if the amount of bleaching gel is reduced or discontinued the procedure. Ask your dentist if the problem persists.

White spots may appear on the teeth during procedures. These are caused by transient demineralization and disappear again.

Warning!

Avoid contact with the eyes, skin and mucosa, in contact wash with plenty of water. If irritation persists, ask for medical advice. Wash hands thoroughly after handling, wear protective gloves, clothing, eye protection, and face protection.

Manufacturing Responsibility

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

Storage : Store well closed in dark and dry place at 4-10°C, protected from direct sunlight, and heat sources. Do not freeze. Don't use after the expiry date. Dispose of contents/bottle to as required by national regulatory requirements. Keep out of reach of children!

Packaging : Syringe 3.3g

10% REF 4050, 16% REF 4055, 22% REF 4060